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mardi, le 24 mars 2020

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---

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**DELIVERED BY EMAIL**  
Tuesday, March 24, 2020

**ATTENTION : All QUSAC/GBH team members**

**SUBJECT : Disinfecting your N95 mask**

Dear team member,

The following steps are approved for the sanitization of the N95 mask so that it may be re-used and provide the expected protection in the same manner as a new n95 mask. When you wear a mask, it is important that you only remove the mask when you are able to sanitize the mask for 30 minutes and then place back into the Ziplock bag provided in you Personal Protection Kit.

A mask can be sanitized in your home oven for 30 minutes at a temperature from 160F (71C) and not exceeding 200F (93C). The following steps must be followed for using your oven at home:

1. Arriving at home:

- I) Remove your mask and place in your oven.
- II) Turn on the oven to a temperature from 160F (71C) and not exceeding 200F (93C)
- III) Once your oven reaches the temperature setting wait another 30 minutes.
- IV) Once the 30 minutes has passed turn the oven off and open the door.
- V) Wait for 30 minutes and remove the N95 mask and place in provided Ziplock
- VI) Your mask is now ready for use again.

2. Arriving at work:

- I) Remove your mask and place in the provided oven (oven will remain on at all times and no temp adjustment will be required).
- II) Wait 30 minutes and return to the oven and remove you mask
- III) Place your mask in the provided Ziplock bag
- IV) Your mask is now ready for use again.

If you follow these recommendations when ever you leave your house or work, you will dramatically reduce your risk of contracting the COVID-19 virus.

---

*Please be sure to wash your hands immediately upon arrival at home or work.*

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